



SomGauge is the first gauge of its kind to allow for accurate calibration of vertical dimension with its adjustable ramp, leading to improved device integrity.

EASY TO USE & CLEAN

Ergonomic and hefty handle with clear protrusion indicators, which is autoclavable.

AVOIDS PREMATURE CONTACT

Adjustable ramp ensures even contact across the occlusal table, which will reduce chair time.

CAPTURES MORE OF THE VERTICAL

At least 3mm of acrylic coverage, leading to a stronger device, less prone to fracture.

HOW TO STEPS

All devices are meant to be titrated from 50% to 60% of a patient's maximum protrusive range. Therefore, our devices are designed to be fabricated in a protrusive position.

1. A minimum of 3 mm of interincisal distance is required throughout the entire upper and lower arch, including the mesial half of the last available posterior (terminal) molar, or the distal ridge of last available molar if full coverage is desired.
2. The bite should be solid without flexing or rocking on the bite fork.
3. Coverage of the VPS medium should be sufficient to engage the teeth with minimal coverage of tissue.
4. For most patients, 60% of maximum protrusion is suitable for a starting position. The SomGauge will allow the patient to find this starting point with ease and accuracy.
5. Once a starting position is obtained, the patient should hold in that position for 2-3 minutes. The dentist should seek feedback on comfort, adjusting as necessary.

Tip: When taking your bite, make sure the skeletal midlines line up.
For more SomGauge tips check out this video [here](#) or scan the QR code.

ORDER THE SOMGAUGE KIT ON SOMACCOUNT.COM

