



Cleaning Instructions

SomTabs is recommended by SomnoMed® for the gentle and thorough cleaning of SomnoMed® devices. SomTabs thoroughly clean the hard-to-reach areas of your device.

Always clean your device in the morning immediately after having removed it from your mouth!



Dissolve a SomTabs tablet in sufficient amounts of warm water in the travel case (approx. 35 °C / 95 °F).

Never use hot water as hot water damages your device and will invalidate the warranty.



Carefully place your SomnoDent® device in the travel case. All parts must be completely immersed in the cleaning solution.



After about 15 minutes, remove the device and small parts from the travel case. **Make sure not to lose any small parts!** Empty the solution from the travel case.



After having removed the device from the travel case, thoroughly brush it with a tooth brush and rinse it under cool running water.



Clean the metal components of the SomnoDent® Flex, Edent, Classic, or Herbst devices with a cleaning brush or a toothbrush.



Dry all parts of the device and the travel case carefully and thoroughly using a clean towel.



When not in use, the device should be stored as per your SomnoMed® device instructions for use.



Important:
Keep your device away from direct sunlight.

Caution:

Do not use toothpaste, as it contains abrasives, and may damage the device.



Do not put SomTabs tablets into the mouth.



Do not swallow the cleaning solution and do not use it as a mouthrinse.

Contents:

Sodium Bicarbonate, Potassium Caroate, Citric Acid, Sodium Citrate, Sorbitol, Sodium Lauryl Sulfate, Sodium Percarbonate, CI 73015.

Ordering SomTabs refills:

Make sure you never run out of SomTabs to clean your device. Order your SomTabs refills in advance.

Simply fill in the order form and send to SomnoMed®, or place your order online by logging on to the SomnoMed® website at www.somnomed.com



SomTabs 1
AM-AUS-ASIA



SomnoMed AG Europe, CH-8052 Zurich, Switzerland, T+800 766 66 633; www.somnomed.eu
SomnoMed Inc., Frisco, TX 75034, USA, www.somnomed.com
SomnoMed Ltd., Cross West HSW 2065, Australia, www.somnomed.com.au